

RECOMMENDED TORQUE OF THE SCREWS FOR PORSTHETIC PARTS

	TEMPORARY RESTORATION					FINAL RESTORATION												
	iphysio Profile Designer	Healing abutment	Direct temporary abutment	Non-rotational temporary abutment	Rotational temporary abutment	Direct clip abutment	Straight or angulated abutment	Customized abutment	Base castable abutment	Customizable abutment	Esthetibase interfaces non-rotational	Esthetibase interfaces rotational	Straight Tetra abutment	Angulated Tetra abutment	Straight Plural abutment	Angulated Plural abutment <i>Discontinued</i>	O'ring abutment	Equator abutment
Tightening torque (N.cm)																		
- Abutment	15	10	25	25	25	35	25	25	25	25	25	25	35	25	35	25	35	25
-													20	20	25	20		
Restorations																		
- Single-unit	x	x	x	x		x	x	x	x	x	x							
- Plural	x	x	x		x	x	x	x	x	x		x	x	x	x	x		
- Complete		x			x	x	x	x		x		x	x	x	x	x	x	x
Prosthesis																		
- Cemented			x	x	x	x	x	x	x	x	x	x						
- Screwed on direct abutment				x	x		x	x	x	x	x	x						
- Screwed on abutment													x	x	x	x		
- Removable																	x	x
Materials																		
- Titanium	x	x	x	x	x	x	x	x		x	x	x	x	x	x	x	x	x
- Gold alloy									x									
- Chrome-cobalt									x									